Multipurpose Solution with aloe vera for silicone hydrogel lenses users. Protein and lipid disinfection (effective against bacteria, fungi and acanthamoeba). Rinsing - storing - wetting - lubricating.

Instructions for use:

1. Before handling your lenses, wash your hands with soap and water.
2. Place your lens on the palm of your hand and rub it gently for 10 seconds with AVIZOR ALVERA.
3. Rub both sides of the lens for at least 5 seconds using AVIZOR ALVERA.
4. Place your lenses in the lens case, and be sure to cover these with AVIZOR ALVERA. Close the case and shake it gently for a few seconds.
5. Keep lenses at least 6 hours in AVIZOR ALVERA (or overnight) to grant an adequate disinfection.
6. Once the disinfection process is done, you may rinse them previously with AVIZOR ALVERA. You may also allow it to air dry.
7. Rinse the lens case with AVIZOR ALVERA and allow it to air dry.

Last revision date: 2012/01

Precautions:

- Once opened, the bottle should not be used after 30 days. The bottle will closed after use.
- Do not use the product if the safety seal is missing or broken.
- You may store your lenses up to 30 days in the closed lenscase
- Do not ingest the solution and keep away from children's reach.
- Keep at room temperature.
- Do not allow the dropper to come into contact with the lens.
- In case of irritation, consult your contact lens specialist.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not use if you are allergic to any of the components.
- Never use tap water to rinse your lenses or lens case.
- Use after the expiration date.
- Discard the solution in the lenscase after each use.
- Your specialist will be able to recommend additional products or changes in cleaning procedures in line with your tear evaluation and the kind of lenses you use and their replacement frequency. Always follow your professional's instructions; and never change them without first consulting him/her.

Composiçao: Poloxamer, EDTA, Aloe Vera, Polihexanida 0.0002% in a buffered, isotonic and sterile solution.

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