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gb USAGE:
Lacrifresh Aloe is a preservative-free multi-dose product specially intended to relieve the symptoms of mild eye irritations caused by pollution, dry or dusty environments, smoke, prolonged use of contact lenses, chlorinated water (swimming pools) and other irritant agents.

It is also indicated for the wetting and lubrication of contact lenses. For contact lens wearers, it provides excellent lubrication, making it easier to insert the lens, improving visual comfort and the wearing of the lens. This product is compatible with all kinds of contact lenses.

INSTRUCTIONS FOR USE:
1. Wash, rinse and dry hands before using this product. 2. Take a bottle of **Lacrifresh Aloe**, pull the safety seal anticlockwise and remove the protective cap. 3. Direct use: Apply one or two drops onto each eye and blink several times. 4. Use with contact lenses: Apply one or two drops into the internal part of each lens before placing on the eye. 5. After each administration, replace the protective cap. 6. Use the product according to your needs, up to 8 times a day. Last revision date: 2017/03.

PRECAUTIONS:
• Do not ingest the solution, and keep it out of children's reach. • Do not use the product if you are allergic to any of its ingredients. • If you experience ocular irritation or an adverse reaction, please consult a specialist. • Once the bottle is opened, use the product for a maximum of 90 days. • Ensure that the tip of the bottle does not come into contact with the eyes, or any other surface, to avoid contamination. • Do not squeeze the bottle with the protective cap on. • Do not use the product if the safety seal is broken or missing, or if the bottle is damaged. • Store the product at room temperature. • Do not use the product after the expiry date. • Do not use the product in combination with a pharmaceutical treatment or any other product that is applied on the ocular surface, as the functions of both could be altered. • Keep the bottle tightly closed after each use.

COMPOSITION: Aloe vera, hypromellose, essential electrolytes (Cl⁻, Na⁺, Ca²⁺, K⁺, Mg²⁺) in aqueous, buffered and isotonic solution.

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